

Article:	Main particle filter
Article number:	239000
Material:	Nylon 12 (PA12)
Total weight:	≈ 30 g
Connections:	Inlet: Ø30 mm, Exterior Ø34 mm. Outlet: Ø34 mm, Exterior Ø40 mm.
Connect to:	240000 flowmeters or 237001 Expiration outlet.
Length:	88,50 mm
Lifespan:	The lifespan of the main particle filter is estimated to be 12 months after which the filter is exhausted and needs to be replaced. The main particle filter may need to be replaced at shorter intervals if the equipment has a high degree of use or if large quantities of smaller particles have been sucked up and caused lower flow or noise.
Cleaning:	<p>Wipe the outside of the main particle filter with disinfectant if necessary. The main particle filter is not in contact with patients, so no need for daily cleaning of the inside is required. If larger particles are sucked into the main filter, such as compresses or the like, remove them and the filter can continue to be used.</p> <p>Recommended disinfectants should contain isopropyl alcohol or ethanol (max. 70%) and surfactants to dissolve dirt</p> <p>If contamination, the main particle filter must be replaced with a new main filter.</p>



Accessories:	240000 Flowmeters
---------------------	-------------------

